



ResMed AirView™



Global



Americas



EMEA



Add Interpretation Notes

Description:

You can add your interpretation to the diagnosis that will also appear on the patient's diagnostic report.

To add interpretation notes to a diagnosis:

1. Open a patient's record from the **Diagnostic patients** list.
2. From the **Diagnostic** tab, click **Add interpretation** under the **Reports** panel.
3. Enter an interpretation into the report.
4. Click **Save**.

Note: *You can edit the interpretation by clicking **Edit** before you sign the report.*