



Associate an ApneaLink Air Device to a Patient Record

Description:

When AirView is unable to find a home sleep test (HST) device associated with a patient record, you can link the device to a patient record via the device association feature in AirView.

To associate an ApneaLink Air device to a patient record:

1. Open a patient's record from the **Diagnostic patients** list.
2. From the toolbar click **Associate device**.
3. Connect the ApneaLink Air device into your computer with a USB cable and click **Continue**.

Note: AirView detects the HST test device and automatically downloads the data.

4. Click **Associate**.

Note: Associating a device will delete all existing data from the HST device.