



Change Prescription Settings Remotely

Description:

You can make changes to a patient's therapy settings from within AirView. A physician should be consulted for any prescription changes.

To change therapy settings using remote monitoring:

1. From the **Prescription** tab, click **Edit settings**.
2. Select preferred changes from the down-down lists.
3. If the patient is connected with a communication module:
 - Click **Send settings to device**. Selecting this option wirelessly updates the patient's therapy settings.
Note: *Incorrect settings can cause incorrect therapy to be applied to patients. Ensure any script changes are approved by the patient's doctor.*
 - Check that the new settings are displayed on screen.
 - Click **Yes, change settings** to proceed with the change. The status is updated with Changes pending. This status will change to No changes pending once the settings are written to the device.

Notes:

- Changes in the settings may take some time to complete. You may also need to refresh the screen to get an updated status.
- If the device settings update is unsuccessful, an error message is displayed. For more information on error messages, refer to the troubleshooting guide.

All prescription changes are displayed in the Logs screen for future reference.