



Change Prescription Settings Using a Data Card

Description:

You can make changes to a patient's therapy settings from within AirView. A physician should be consulted for any prescription changes.

To change therapy settings using a data card:

1. From the **Prescription** tab, click **Edit settings**.
2. Select preferred changes from the down-down lists.
3. Click **Save settings to card**.
4. Click **Next**.
5. Insert the patient's data card and click Continue.
6. Select the data card from the list to write settings to.

Note: Ensure that you select the correct data card as AirView will erase all existing data in the data card.

7. Click **Continue** to confirm.
8. Check that the Status changed to **Settings written to card**.

Notes:

- Therapy settings are updated only after inserting the data card to the patient's device. Until then, any changes made will not be applied and shown in AirView.
- Always check that the data card is correctly formatted and/or not locked for editing.

All prescription changes are displayed in the Logs screen for future reference.