



ResMed

AirView™




Global



Americas



EMEA

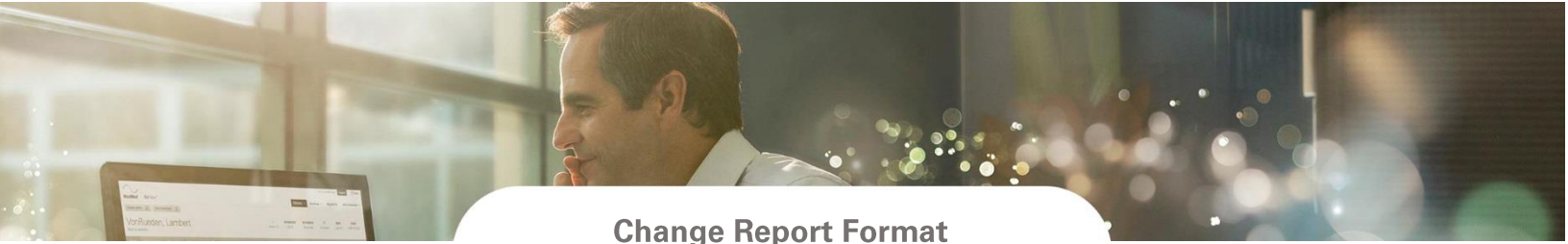

Change Report Format

Description:

You can change the patient's Diagnostic report format from Standard which shows a summary data and interpretation notes only, to Detailed which adds additional graphs to the diagnostic report. The Diagnostic report format cannot be changed once an electronic signature has been added to the diagnosis.

To change the diagnostic report format:

1. Open a patient's record from the **Diagnostic patients** list.
2. In the **Reports** section, click **Change format**.
3. Select the **Detailed (data and graph)** option and click Save.

Notes:

- *The diagnostic report format cannot be changed once a physician has added their electronic signature to the diagnosis.*
- *The report type selected will become the default report type for new Diagnostic reports.*