

ResMed

AirView™




Global



Americas



EMEA

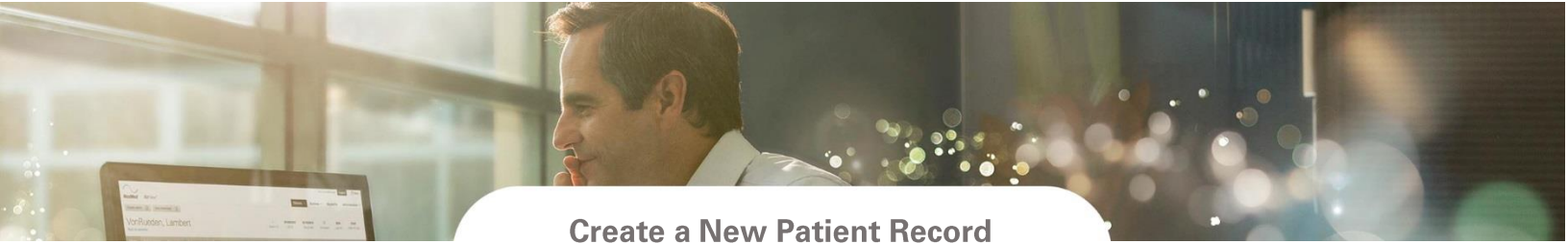

Create a New Patient Record

Description:

You can enter new patient details into AirView. Once a patient's record is created, you can download home sleep test device data, generate a diagnostic report, and associate a device to a patient.

To create a patient record:

1. From the **Diagnostic patients** screen, click **New patient**.
2. Enter the following details:
 - **Clinical details**
 - **Clinical user:** Defaults to currently logged-in user. Click the drop-down arrow to select another user who will accept the patient.
 - **Physician:** Select a physician who is linked to share the patient record for interpretation.
 - **Patient details**
 - Name
 - Date of Birth
 - Patient ID (optional)
 - **Healthcare details** (optional)
 - **Contact details** (optional)
 - **Additional details** (optional)
3. Click **Save**.

Note: It is recommended to immediately associate the device to the patient after creating the patient record to ensure that the auto search will find the patient during device download.