



ResMed

AirView™




Global



Americas



EMEA

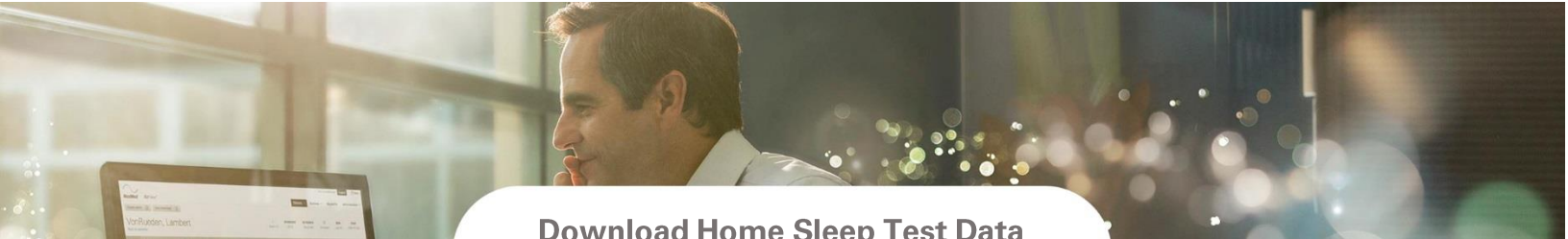

Download Home Sleep Test Data

Description:

Once the HST device is returned to your location, it is time to download the data.

To download home sleep test device data:

1. From the **Diagnostic patients** screen, click **Device download**.
2. Connect a compatible HST device into your computer with a USB cable and click **Continue**. AirView detects the device and automatically looks for an active patient with matching serial number. If no patient is found, you can search for a patient or create a new patient.
3. If the correct patient record is displayed, click **Download**. Ensure the correct patient has been selected, as this action cannot be undone. The device download starts automatically and a confirmation message is displayed. For Nox T3 patients, the Diagnostic panel will display to confirm data. Click **Save** to continue.
4. If the wrong patient record is displayed, click **Search active patients**.

Note: *If the device is already assigned to another patient outside of your organization, you will not be able to reassign the device. Contact your local technical services team for assistance.*