



Electronically Sign a Diagnostic Report

Description:

Interpreting Physician users within AirView can add an electronic signature to the diagnosis that will also appear on the patient's diagnostic report.

To add an electronic signature to a diagnosis:

1. Open a patient's record from the **Diagnostic patients** list.
2. Enter your password under the **Signature** panel and click **Sign**.

Note: *Your e-signature password is the same as your login password. Only users with 'Interpreting physician' permissions can electronically sign diagnostic reports*

To remove an electronic signature:

1. From the **Diagnostic** tab under the **Signature** panel, click **Remove signature**.
2. Confirm by clicking **Yes, remove signature** on the dialog box.