



Wireless Patients Page

Description:

The Wireless patients screen lists only the patients with wireless monitoring. The Wireless patients page displays a summary of therapy from the 10 most recent days of available data for patients that are transmitting data wirelessly.

To access the Wireless patients page:

1. From the Patients menu, select Wireless.
2. Each colored box provides a summary of the patient's therapy from the 10 most recent days.
3. Click on each colored box to see more information.

Note: Data is transmitted to AirView daily when wireless coverage is available.

Icon guide:

Icons		
Daily compliance status		
Icon legend:		Top—AHI / Bottom—Leak
		<ul style="list-style-type: none"> • Below threshold (green) • Above threshold (yellow) • Data not supported (white)
		Overall colour—Usage
		<ul style="list-style-type: none"> • Above threshold (green) • Below threshold (red)
Usage above threshold	Usage below threshold	
		
		Leak above threshold
		Leak data not supported
		AHI above threshold
		AHI data not supported
		AHI above threshold, leak data not supported
		Leak above threshold, AHI data not supported
		AHI, leak above threshold
		AHI, leak data not supported
		Device not used
		Monitoring—no data received yet
		Not monitoring—no data available
		Device has fault—no data available