


# Diagnostic Scoring Guide

English

## Data signals dashboard

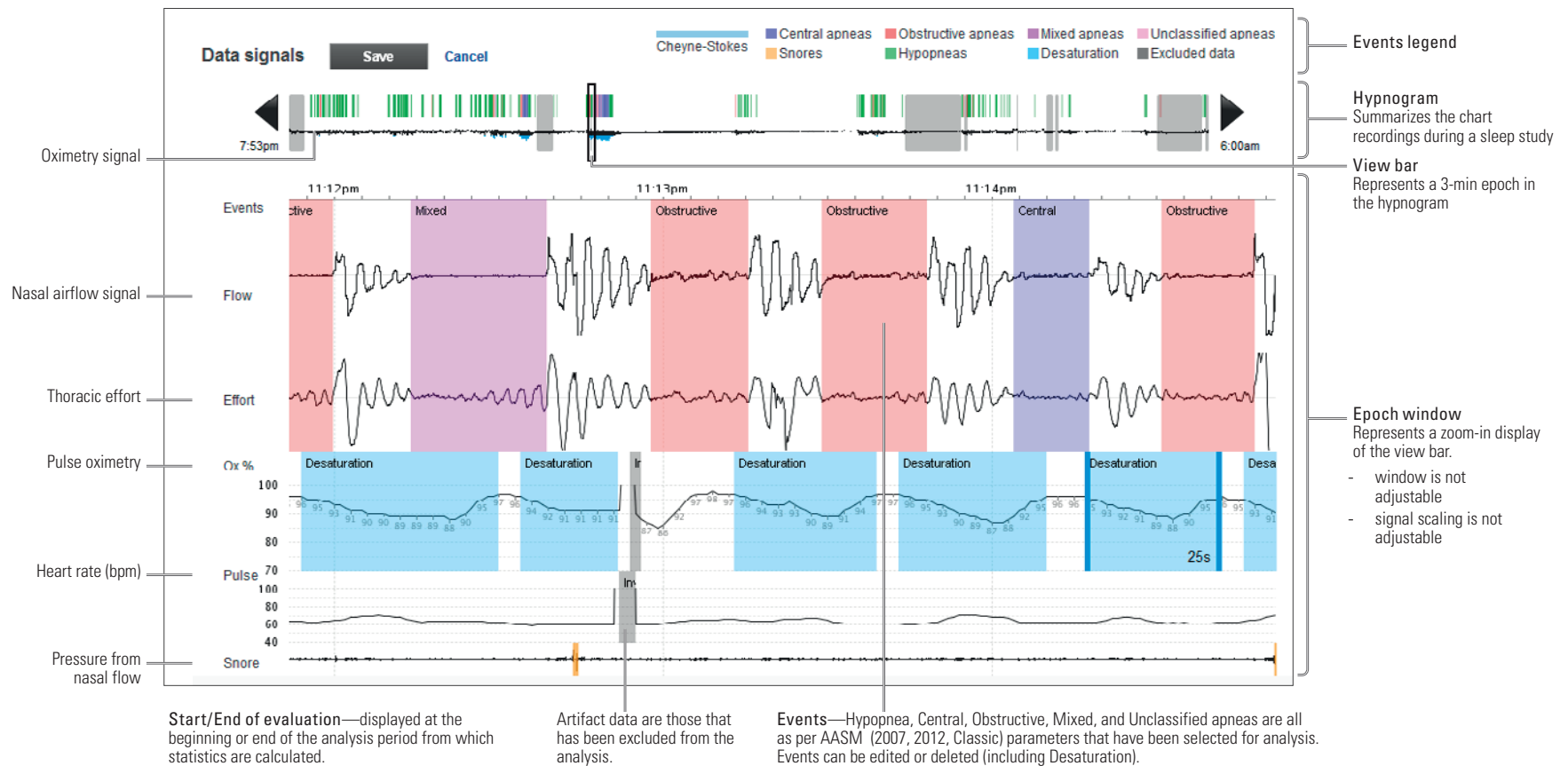

### To navigate:

1. Click the left or right arrow on the hypnogram to move the display to a 3-minute epoch.
2. Press CTRL + left or right arrow key on the keyboard to move the display to a 90-second increment.
3. Click and drag the view bar within the hypnogram to display any time period.




## Step-by-step manual scoring


Data is auto-scored when data is downloaded into AirView. If required, you can manually score the raw data signals which can override the auto-scoring.

To manually score:



1. Select **Raw data signals** from the Diagnostic tab of a patient's file.
2. Manually change data as required:

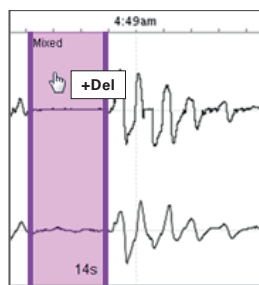


To delete, hover over the event and press **Delete** on the keyboard.

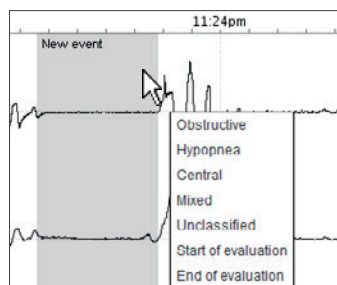

To insert, click the left mouse button at the start of the event and drag the cursor to the end of the event. Select the event type to finish.

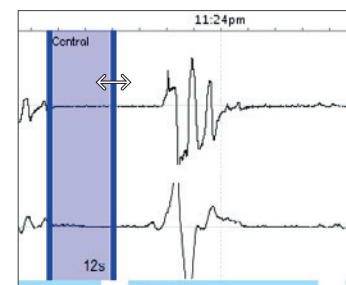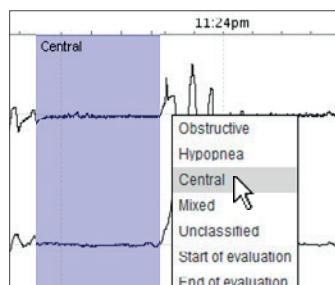

To change an event duration, move the cursor to the beginning or end of an event and drag the left-right arrow to the desired position.

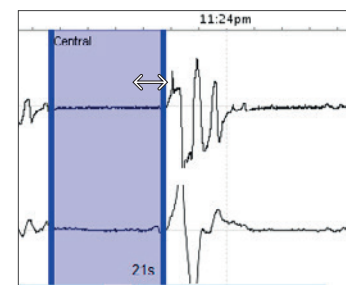

3. Click **Save**. Statistics on the diagnostic screen are automatically updated.

To reset to auto-score:

1. From the **Statistics** panel under the **Diagnostic** tab, select the AASM analysis guideline from the drop down menu.
2. Click **Reanalyze**.