

Marking Patients as Reviewed

Quick Reference Guide



Why?



By marking patients as reviewed in each U-Sleep™ Action View group, you create a cleaner dashboard. This allows you to:

- Minimize your workload by focusing on patients that may require additional coaching.
- Identify situations where issues in therapy are reoccurring.

How?

- 1 Sign in to U-Sleep.
- 1 Click one of the **Action View** groups that displays at least one patient.
- 2 Complete your necessary action with the patient.
- 3 Check the patient's name.
- 4 Click **Patient Reviewed**.
 - If the patient is in more than one group, a pop-up will display asking if you want to mark the patient as reviewed for the other groups as well.
 - Check off any other groups that you want to mark the patient as reviewed from.
 - Click **Reviewed from groups**.
 - Click **OK**.

The patient will be removed from your working patient list, allowing you to move onto the next patient.

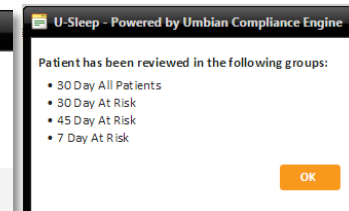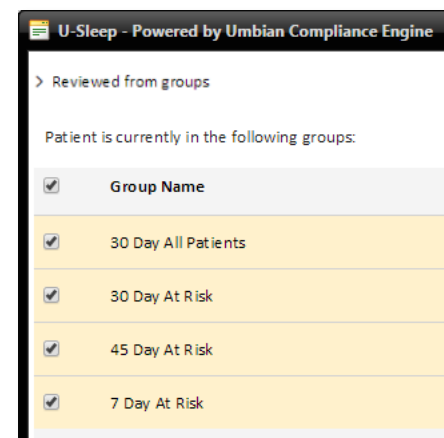

Distributed by ResMed Halifax, ResMed 38 Solutions Dr., Halifax, NS B3S 0H1 Canada +1 (877) 242 1703 Ext 369235. See www.resmed.com for other ResMed locations worldwide. U-Sleep is a trademark and/or registered trademark of the ResMed family of companies. © 2016 ResMed. RH-102019/1 2016-04

U-Sleep Support
support@u-sleep.com | 1-800-424-0737

