

Patient Compliance and Therapy Report


Quick Reference Guide



Why?

Perform this procedure if you want to create the AirView™ Compliance and Therapy report for a patient.

How?

- 1 Sign in to U-Sleep™.
- 2 In U-Sleep, click the **People** tab.
- 3 Search for the patient that you want to create a report for.
- 4 To view the report options:
 - Click the report icon () to the right of the patient's username.
- OR
 - Click the patient's name and then click **View Reports** in the **Tasks** menu.
- 5 In the **Select Report** drop-down list, select **Compliance and Therapy Report**.
- 6 Click **Next**.
- 7 Select a date range (for custom date range, the maximum is 90 days).
- 8 Click **Run Report**.
- 9 If you are not already signed in to AirView, you will be prompted to enter your AirView username and password.
- 10 Click the patient's name in the search results to display report.

Reports

Select Report: Compliance and Therapy Report

Step 1: Select Patient

Patient: Luis Garcia Select...

Next >

Step 2: Select Date Range

Reports

Select Report: Compliance and Therapy Report

Step 1: Select Patient

Step 2: Select Date Range

Available Dates: Apr 11, 2015 - Dec 08, 2015

Last 30 Days


Last 14 Days

Last 7 Days

First 30 Days

Custom Date Range

Start Date: Nov 08, 2015 

End Date: Dec 08, 2015 



< Previous Run Report

Patient Compliance and Therapy Report

Quick Reference Guide



U-Sleep pulls the **Compliance and Therapy Report** from AirView.

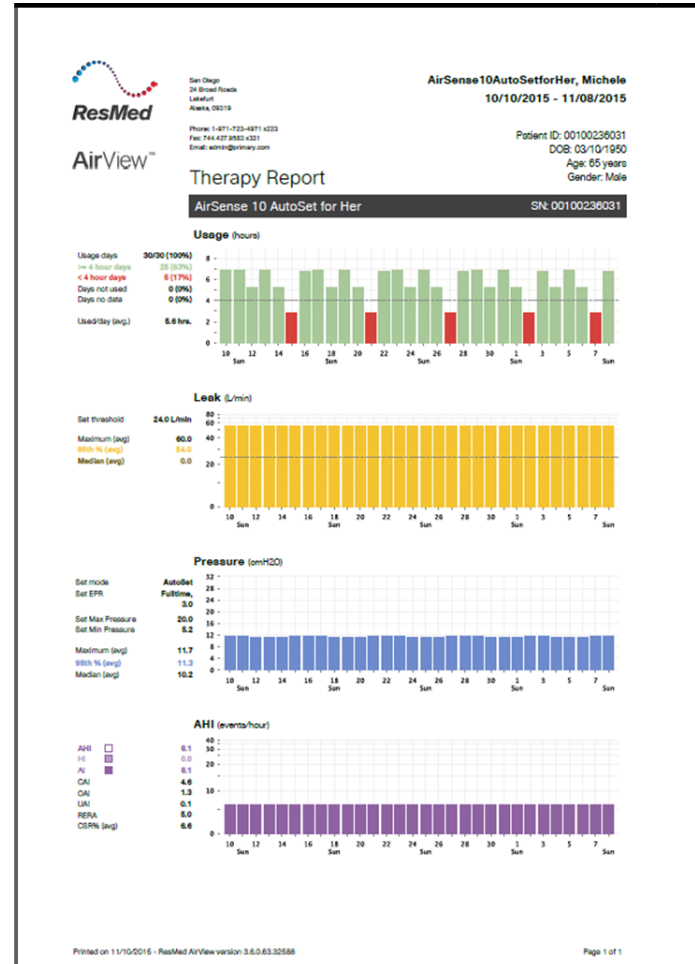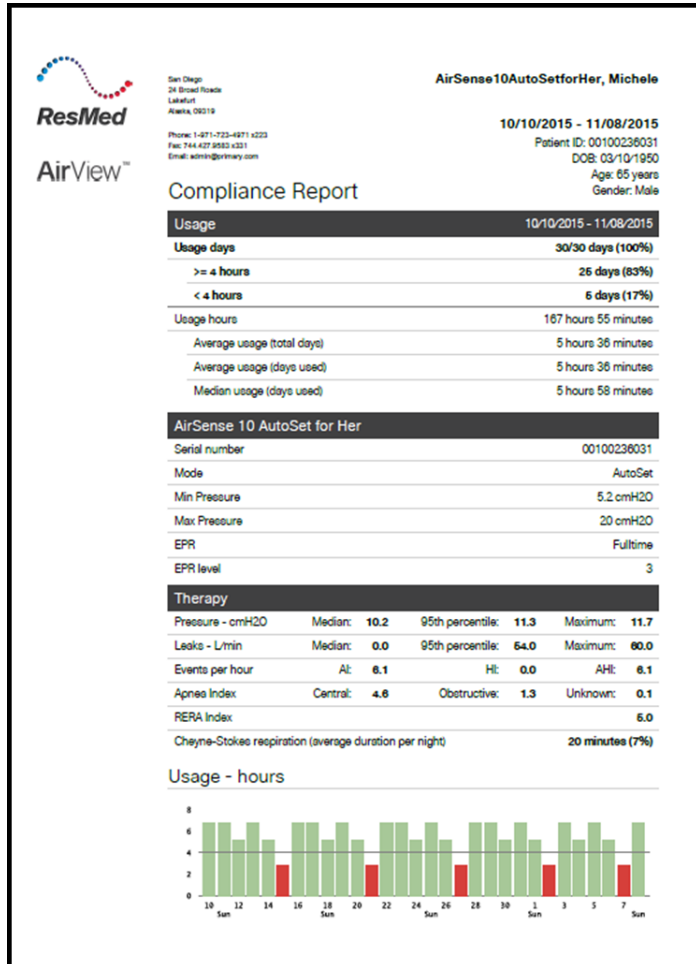

Distributed by ResMed Halifax, ResMed 38 Solutions Dr., Halifax, NS B3S 0H1 Canada +1 (877) 242 1703 Ext 369235. See www.resmed.com for other ResMed locations worldwide. U-Sleep and AirView are trademarks and/or registered trademarks of the ResMed family of companies. © 2016 ResMed. RH-102021/1 2016-04

U-Sleep Support
support@u-sleep.com | 1-800-424-0737

