



## View and Print Reports

### **Description:**

Reports can be produced for each patient within AirView

**Note** - These reports are:

- **Compliance report** - displays a summary of the patient's compliance data.
- **Therapy report** - displays a graphical summary and statistical information of the patient's therapy data.
- **Compliance and therapy report** – displays a combined summary of the patient's compliance and therapy report.
- **Detailed report** - displays a patient's detailed data.

All reports are generated in Adobe PDF format which you can either save as a file or print, if required.

### **To quickly generate a compliance report:**

1. From the **Patients** menu, select **All therapy**.
2. Locate the patient record.
3. Click the **green checkmark** next to a compliant patient's name.
4. The report is displayed in a new tab in Adobe PDF format which you can either save as a file or print, if required.

### **To generate any report:**

1. Open the patient's file.
2. From the toolbar, click **Create report**.
3. Select the desired report type from the drop-down list.
4. Select preferred time period.
5. Click **Continue**. The report is displayed in a new tab in Adobe PDF format which you can either save as a file or print, if required.